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CONTRE-INDICATIONS
Do not use Cavex Bite&White ExSense in cases of allergies to mint aroma or any of the ingredients.

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AVEC GOUTTIÈRE :
Dispense a small ribbon of Cavex Bite&White ExSense across the entire tray. Insert tray and gently rotate tray left and right to assure gel is distributed across teeth. Gently bite down on the tray and leave gel in place for 10 minutes. Remove tray, spit out any excess gel and rinse mouth with water. Rinse tray with cold water.

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GB Instructions for use

DESCRIPTION
Cavex Bite&White ExSense is a water-based gel with mint aroma that contains hydroxyapatite and potassium nitrate. Cavex Bite&White ExSense controls sensitivity and enhances whiteness of the teeth by hydroxyapatite remineralization.

PACKAGING
Tube 42 ml – 50 gram (sufficient for approximately 20 treatments)

COMPOSITION
Key ingredients:
Hydroxyapatite 2%, Potassium nitrate 0,1%

INDICATIONS FOR USE
■ After in-office and/or at-home tooth whitening.
■ After professional tooth cleaning.
■ Prevention and control of sensitivity.

CONTRA-INDICATIONS
Do not use Cavex Bite&White ExSense in cases of allergies to mint aroma or any of the ingredients.

HANDLING PROCEDURE
■ Prepare and clean the teeth using brush and floss.

APPLICATION:

WITHOUT TRAY:
Wash hands thoroughly with soap and water. Dispense a small amount of Cavex Bite&White ExSense onto a clean finger. Gently apply gel on all surfaces of the upper and lower teeth. Allow gel to remain on tooth for 10 minutes. Spit out any excess gel and rinse mouth with water.

WITH TRAY:
Dispense a small ribbon of Cavex Bite&White ExSense across the entire tray. Insert tray and gently rotate tray left and right to assure gel is distributed across teeth. Gently bite down on the tray and leave gel in place for 10 minutes. Remove tray, spit out any excess gel and rinse mouth with water. Rinse tray with cold water.

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WARNING & FIRST AID INSTRUCTIONS
In case of contact with the eyes: flush thoroughly with plenty of water and seek medical attention.

Do not use the material after the expiry date.

STORAGE ADVICE
Store in a cool place, dry place (frost-free) and protected from direct sunlight.

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Your impression is our concern
Made in The Netherlands

The application, use and processing of the products are beyond our control and, therefore, entirely your own responsibility. Should, in spite of this liability, be established for any damage, it will be limited to the value of the goods delivered by us and used by you. We will, of course, provide consistent quality of our products within the scope of our General Conditions of Sale and Delivery.

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