REGENO

Plasma Liquid

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Plasma Liquid

Instructions for use Plasma Liquid® Nasal Spray Gel

Product properties: Plasma Liquid® Nasal Spray Gel is a pure mineral gel with physical effects for moistening and cleaning the nasal mucous membrane. Dried nasal mucous membranes create a favourable environment for bacteria, fungi and viruses. As a preventive measure, it is therefore advisable to sufficiently moisten the nasal cavity and thus strengthen the local immune system of the mucous membrane.

The well adhering consistency of Plasma Liquid® nasal spray gel achieves an effective moistening of the nasal mucosa. In addition, its low surface tension ensures particularly thorough cleansing. Together with the moisturizing effect, this mechanical reduction of the germ load has a harmonizing and strengthening effect on the nasal mucous membrane.

Plasma Liquid® Nasal Spray Gel has a soothing and calming effect. It can be used for prevention and supportive application in case of inflammations due to infections of the upper respiratory tract (banal cold) and in case of longterm allergic reactions (e.g. hay fever). It can also be used as an adjunct for diseases of the upper respiratory tract that are difficult to treat systematically, e.g. chronic sinusitis.

After application, the composition decomposes into the natural components water and salt. Plasma Liquid® Nasal Spray Gel contains no greasy or oily components. It is also free of alcohol and without propellant gas. The composition is based on an electrochemically activated mineral salt solution with a silicate as a gel former. The contained preservative sodium hypochlorite is also used by the body's own immune system. It has a broad spectrum against bacteria, fungi and viruses, which enables the safe application of the product after opening and during the entire shelf life. The skin and mucous membrane compatibility of Plasma Liquid® Nasal Spray Gel is dermatologically confirmed.

Application: Shake well before using! Plasma Liquid® Nasal Spray Gel can be used 3 to 5 times a day with 1 to 2 sprays per nostril. When used as intended, slight irritation (e.g. tingling or burning) may occur in isolated cases. These usually subside quickly. For reasons of hygie-

ne each product should only be used by one person.

Application Restrictions: Plasma Liquid® Nasal Spray Gel should not be used in case of known incompatibilities with any of the ingredients. Do not use simultaneously with other solutions, sprays or ointments. Only for moistening and cleaning the nasal mucosa. For use during pregnancy, breastfeeding and in children under two years of age, consult your doctor or pharmacist beforehand. For children up to 12 years of age, the application should be supervised by an adult.

If acute inflammation worsens or there is no improvement after 3-5 days, you should consult a doctor. **General information:** Avoid eye contact. Keep safe from children. Use only faultless and undamaged bottles. Protect from direct sunlight. Do not use Plasma Liquid® Nasal Spray Gel beyond the expiration date shown on the bottle and do not use for more than 12 weeks after opening.

Ingredients: Aqua, <0,06% sodium hypochlorite, lithium magnesium sodium silicate.

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